

# Caring for spencer

How often you wipe down spencer depends on the frequency of visitors, including home health aides or other caregivers.

## RECOMMENDED CLEANING

Using a 70% isopropyl alcohol wipe, or Clorox® Disinfecting Wipe to gently wipe the hard, nonporous surfaces of the spencer device. This includes the front display screen, the top with the lid closed, the sides and the back of the spencer device. Wring excess liquid from the wipe before using on spencer. Let spencer air dry.

## A FEW "DON'TS"

- Don't use bleach.
- Avoid getting moisture in any opening – including the mouth of the spencer device – where you receive your packaged medications.
- Don't use aerosol sprays, bleaches, or abrasives; and don't spray cleaners or disinfectants directly onto the spencer device.
- Never clean or wipe the spencer refill package, or the individual medication packs. This could remove or damage the identifying barcodes.

